



Järven Bed Protector

The Järven Bed Protector has a high absorption capacity and is available in four different models, all made of easy to clean synthetic fibre. Two of the models can also be used as a draw sheet as the underside consists of a slidable material.

The bed protector is available in sizes that fit children's beds.

The Järven Bed Protector should be washed in 60°C short cycle and in 90°C for disinfection, for 7 minutes, if possible. Drying temperature should be a maximum of 70°C.

Avoid washing the bed protector together with hard textiles and Velcro straps. (alternatively a used laundry washing bag)

Never use antistatic detergent or softener.
Chlorine bleaching is prohibited.

The Järven Bed Protector is made without vinyl and therefore retains its softness after washing. The Järven Bed Protector has a laminated moisture barrier and the different material layers are well merged into one single unit through welding. This prevents the sheet from wrinkling underneath the patient, which can happen when products with loose layers are used.

FACTS

Item no.	Designation
90600	Malin, 75 x 90 cm + wings 50 cm/side
90620	Malin, 100 x 90 cm + wings 50 cm/side
90630	Malin, 200 x 90 cm + wings 50 cm/side
90700	Linn, 75 x 90 cm + wings 50 cm/side. Underside with sliding properties 75 x 75 cm
90750	Linn, 100 x 90 cm + wings 50 cm/side. Underside with sliding properties 75 x 90 cm
90740	Wilma, 85 x 150 cm. Underside with sliding properties 85 x 150 cm
90820	Anna-Maria, 200 x 80 x 13 cm
90822	Anna-Maria, 200 x 90 x 13 cm

Recycling

The bed protectors are fully recyclable and should be sorted as combustible waste.

Cleaning

Cleaning and care instructions can be found in the directions for use.



Please read the instruction manual before using the product.



MALIN

With wings and a banded border around it.




LINN

With wings and a banded border around it. Part of the underside is slidable.



WILMA

With wings and a banded border around it. The entire underside is slidable.

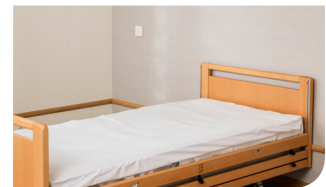

ANNA-MARIA

Underside and short sides made of cotton / polyester. Zipper on one short and one long side.



Järven Health Care

Bromsvägen 3 | SE-891 60 Örnsköldsvik | SWEDEN
Telephone: +46 (0)660-22 28 50 | Telefax: +46 (0)660 - 824 10
www.jarven.se | healthcare@jarven.se